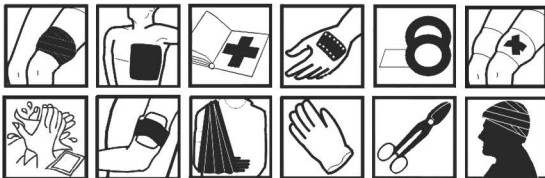
# KIXSAFE®

## AUTO FIRST AID KIT

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Instruction for immediate life-saving measures in case of accidents

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Specification contents DIN 13164


Indice dei contenuti DIN 13164

## A) Giving help and reassurance, emergency call

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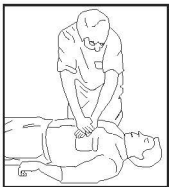---

**It is a moral obligation of every human being to help victims of accidents. Give first aid and care as good as possible.**

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**Have the rescue services called out as soon as possible.**

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---

### **Emergency call**

**Where? Give exact details about the place of emergency!**

**What happened? Give a short description of the accident scene!**

**What number? Tell how many persons are injured. What kind of injuries!**

**What kind of injuries? Specify if there are any threatening injuries!**

**Wait! The operator will finish the call when he/she has all information he/she needs!**

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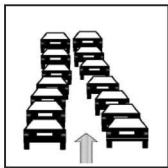
## C) How to act at the site of an accident

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---

**Allow ambulances to pass between the waiting vehicles.**

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**First of all safeguard the site of the accident, put up warning triangle.**

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**Assess the situation and act cautiously and considerately.**

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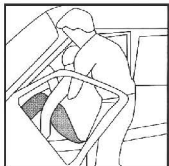
**Address casualty, open the safety belt and grasping his/her hips turn casualty's back towards you.**

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**Get ahold of casualty's forearm passing your arms under his/her armpits.**

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**Drag casualty out of the vehicle and put him/her down in a safe place.**

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## E) Removing a crash helmet

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---

**Open the visor and address casualty.  
If not responsive: (G) Recovery position.**

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**Remove crash helmet by two first aiders  
as shown here.**

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**Check breathing. If casualty is breathing:  
(G) Recovery position.**

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## F) Checking the vital functions

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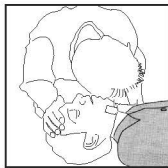
**Patient does not respond:  
(G) Unconscious.**

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**No breath to be heard, no movement of the  
chest: (H) Respiratory failure.**

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**No signs of life like breathing, movement,  
swallowing, coughing: (I) Circulatory failure.**

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---

**If not responsive, check breathing:  
If breathing is present recovery position.**

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**Stretch near side arm and push it under the hip.**

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**Bend near side leg.**

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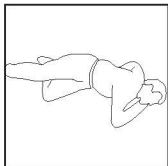
**Get ahold of far side shoulder and hip and turn casualty cautiously on his/her side.**

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**Recline his/her head, open his/her mouth, and bend backward arm.**

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**Keep casualty warm and call for the ambulance.**

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#### Mouth-to-nose respiration

In case of respiratory failure:

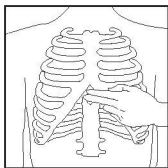
1. Recline head.
2. Keep nose or mouth closed.
3. Give breath into mouth or nose.
4. Turn your head and watch for breath sign.
5. Give 15 breaths per minute.

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**Cardio-pulmonary resuscitation (I + H)**

**Cardio-pulmonary resuscitation:**

**Continue CPR until a doctor is available, applying artificial breathing and chest compression (100/min) at a rate of 2:15.**

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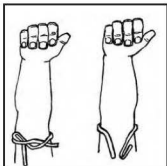
### Sanguinamenti

---

**In case of severe bleedings from the arm, first elevate arm and apply indirect pressure at the upper arm.**

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**Dress severe bleedings from arm or leg by applying a pressure bandage.**

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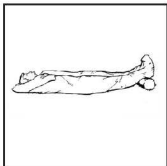
## K) Shock

### Scossa

**Casualties who have a shock are pale, they are cold, and they have a feeble pulse.**

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**Place casualty with his/her legs in an elevated position, reassure and keep him/her warm. There is a rescue foil in the first aid kit.**

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## L) How to position a casualty

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---

**In case of injuries or illnesses in the abdominal region, place casualty with bent knees for relaxation as shown here.**

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**In case of breathing difficulties or cardiac infarct, place casualty with head and chest elevated if he/she is responsive.**

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**In case of head injuries such as a concussion of the brain: place casualty with his/her head elevated if he/she is responsive.**

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**A casualty who has a shock is positioned on his/her back with his/her legs elevated if he/she is responsive.**

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DIN 61634-FB 8

**First aid blanket**

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To protect against  
heat and coldness

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